Protein Check

de	Gebrauchsanweisung
cs	Návod k obsluze
da	Brugsanvisning
el	Οδηγίες χρήσης
en	Operating instructions
es	Instrucciones de manejo
fi	Käyttöohje
fr	Mode d'emploi
hr	Upute za uporabu
hu	Használati útmutató
it	Istruzioni d'uso
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja użytkowania
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sl	Navodila za uporabo
sv	Bruksanvisning
tr	Kullanım kılavuzu

de	4
cs	6
da	8
el	10
en	12
es	14
fi	16
fr	18
hr	20
hu	22
it	24
nl	26
no	28
pl	30
pt	32
ro	34
sk	36
sl	38
sv	40
tr	42

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en - Application

Application

The ProCare protein check is used to verify quality in relation to protein residues at the end of a thermal disinfection reprocessing operation.

Verification is achieved through the way the acid red dye binds specifically to protein.

It can be used for instruments at:

- Local doctor's practices
- Outpatient operating premises
- Hospital clinics
- Centralised and decentralised instrument reprocessing facilities at hospitals

Please note the safety data sheet.

Lower detection limit

Visible discolouration indicates protein concentration > 6.5 µg/cm².

Safety data sheet

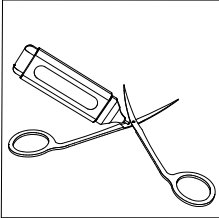
You can find the safety data sheet here:

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Carrying out the test

- Remove the cap from the pen.

The tip of the pen functions as a valve. When the tip is pressed down, dye solution leaks out.

- Place the pen on the surface to be tested and press down on the tip so that dye solution leaks out.

Leave the dye solution to work for 3 minutes.

For a clear result, it is vital to wait the full 3 minutes.

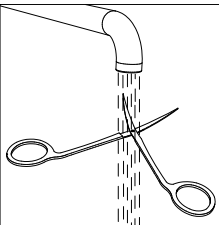
3 min

- Rinse the dye solution under running water for 2 to 3 seconds.
- Check the surface to be tested.

Red discolouration of the surface indicates the presence of protein residues.

Aftertreatment of instruments

No discolouration

- Rinse the instrument thoroughly to remove the dye.

Discolouration

- Clean the instrument one more time.
- Rinse the instrument thoroughly one more time.

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